# Clarity<sup>HCG</sup> Combo Cassettes

## CLARITY Pregnancy Combo Test (Cassette)

One step assay  
Instant visual results  
Accurate detection  
For in vitro qualitative diagnostic use

### INTENDED USE

The **CLARITY Pregnancy Combo Test (Cassette)** is a qualitative immunoassay for the early detection of pregnancy in urine and serum.

### SUMMARY AND EXPLANATION OF THE TEST

This pregnancy test is based on the detection of the human chorionic gonadotropin (hCG) in urine and serum. hCG is a hormone produced by the placenta. In normal subjects, hCG in urine and serum provides an early indication of pregnancy (1). The **CLARITY Pregnancy Combo Test** uses a mouse monoclonal antibody specific to hCG in a one-step lateral flow chromatographic immunoassay to instantly and accurately detect hCG at the level close to or greater than 20 mIU/ml. (WHO 3<sup>rd</sup> IS 75/537)

### PRINCIPLE OF THE PROCEDURE

This assay is a one-step lateral flow chromatographic immunoassay. The test strip in the device consists of a conjugate pad containing mouse monoclonal anti-hCG antibody conjugated to Colloidal Gold, and a nitrocellulose membrane strip containing a test line (T line) and a control line (C line).

### EXPECTED VALUES

This test is capable of detecting hCG at the level as low as 20 mIU/ml (WHO 3<sup>rd</sup> IS 75/537) or the first day of a missed period and no sooner. In normal subjects, hCG in urine and serum provides an early indication of pregnancy. In a 28-day cycle with ovulation occurring at day 14, hCG can be detected in urine and serum in minute quantities around day 23, or 5 days before the expected menstruation. The hormone concentration doubles approximately every 2 days and peaks between 7-12 weeks after the first day of the last menstrual period with a mean concentration of 50,000 mIU/ml. Concentrations as high as 100,000 mIU/ml have been reported in normal pregnancies during the first trimester (1).

### REAGENTS AND MATERIALS SUPPLIED

1. Test Device and Dropper pipette sealed in foil pouch
2. Package Insert

### MATERIAL REQUIRED BUT NOT PROVIDED

1. Specimen collection container
2. Timer or watch

### STORAGE AND STABILITY

Store the kit at room temperature (2-25°C) (35-77°F). Kit contents are stable for at least 18 months or until the date printed on the label, whichever comes first.

Do not freeze the kit as this may cause malfunction of the kit.

Exposing the kit to the temperatures over 30°C (86°F) will reduce the shelf life of the kit. For example, one week at 45°C (113°F) will reduce the shelf life of the kit by 10 weeks.

### SPECIMEN COLLECTION

1. Urine specimen should be collected in a clean container. No centrifugation or filtration of urine is required.
2. Serum specimen should be collected following standard clinical procedure.
3. Specimens may be kept at room temperature for 2 hours, if specimens cannot be tested after collection, they should be stored refrigerated at 2-8°C (35-46°F) for use to 3 days and at -20°C (-4°F) or lower. Do not mix specimens.


### PRECAUTION

1. Do not open the sealed pouch unless ready to operate the assay.
2. Do not use expired kit.
3. Dispose of all specimens and used assay materials in a proper biohazard container.
4. The instructions must be followed to obtain accurate results.

### ASSAY PROCEDURE

1. Refrigerated specimens or other materials must be equilibrated to room temperature before testing.
2. Remove the test device from pouch and place it on a flat surface.
3. Add 4 drops of the specimen to the sample well marked as S on the device.
4. Strong positive results may be observed in 2-3 minutes. Weak positive results may take longer time, up to 5 minutes. Do not interpret the results after 10 minutes.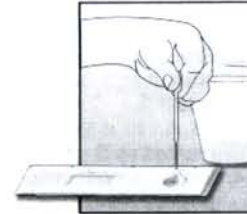

### INTERPRETATION OF RESULTS

1. **POSITIVE:** Both C line and T line appear in the viewing window accordingly, indicating the hCG is detected in the specimen at the level close to or higher than 20 mIU/ml.
2. **NEGATIVE:** Only one line, the C line, appears in the control region, indicating that there is no hCG detected in the specimen and the result is negative. If pregnancy is suspected, repeat the test after 2 or 3 days with new devices and fresh samples.
3. **INVALID:** No band visible in the control region within 5 minutes. Repeat the assay with a new test device.

+ (Positive)

- (Negative)

### OVERALL ACCURACY

The overall accuracy is  $\geq 99\%$

### SENSITIVITY

The sensitivity is 20 mIU/ml.

### SPECIFICITY

The specificity is 100%

### QUALITY CONTROL

#### INTERNAL QUALITY CONTROL

Internal procedural controls are included in the test. A red line appearing in the control region (C) is an internal positive procedural control. It confirms sufficient specimen volume and correct procedural technique. A clear background is an internal negative background control. If the test is working properly, the background in the result area should be white to light pink and not interfere with the ability to read the test result.

#### EXTERNAL QUALITY CONTROL

In addition to your laboratory's standard quality control procedures, it is recommended that a positive and negative external control be tested once within each box.

### QUESTIONS & ANSWERS

- Q:** If I see two bands appear, but one band is darker than the other, am I pregnant?  
**A:** Yes, This should be interpreted as a positive result.
- Q:** Can the result change after standing for a certain length of time?  
**A:** A positive result will not change after the test is completed after 3 minutes. However, in order to prevent any incorrect results, the test results should not be interpreted after 10 minutes.
- Q:** If the test result is negative, what do I do?  
**A:** A negative result means that no pregnancy hormones have been detected and you are probably not pregnant. If your period does not start within a week, you may have miscalculated the day your period was due. Repeat the test.
- Q:** Can I get a false or misleading result?  
**A:** This should not happen if you follow the instructions correctly. However, if you get the absorbent material wet before using the test, then you may not get a reliable result. Certain rare medical conditions can also cause a false result.

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